CZ | Ochranné kalhoty

1 | Dodávky:

- 1 x Ochranné kalhoty
- 1 x Návod k obsluze

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GB | Protective pants

1 | Delivery:

1 x protective pants
1 x Instruction manual

Read the instructions for use completely and carefully! ⁱ

2 | Intended purpose:

The incontinence pants is used to alleviate the effects of mild to moderate incontinence. The suction insert sewn into the incontinence pants absorbs the moisture and processes it.

For private use.

3 | Contraindications / Not suitable: ⁱ

In the case of the following medical conditions, it is only recommended to wear this article after consultation with your doctor:

• Skin diseases/injuries in the affected part of the body, especially in inflammatory symptoms such as swelling, redness and overheating.
• Allergic reactions to the material.

4 | Notes on safe use of the product: ⁱ

Side effects affecting the whole organism are not yet known. Proper application is required. In case of improper use, product liability is excluded.

• The clothing sizes must be observed.

• If your symptoms worsen or you notice unusual changes, consult your doctor immediately.

5 | Application:

The incontinence pants are worn like ordinary underwear.

6 | Cleaning:

Cleaning is done by usual household laundry, according to care instructions. Product can become smaller in the dryer.

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